

# Evita series Remote Pad

Evita Remote Pad –  
The convenience factor in Respiratory Care



- Clinical routine functions accessible from any place around the bed
- Convenient activation of:
  - Alarm silence,
  - Insp. Hold,
  - Exp. Hold,
  - O<sub>2</sub>-suction,
  - Start nebulizer,
  - Alarm Reset
- 3 m cable length
- Insp. Hold during X-ray from a greater distance

## THE CONVENIENCE FACTOR IN RESPIRATORY CARE

Have you ever wondered why you had to walk around the bed several times while performing a suction procedure with your patient?

In most working environments, and even in private life, ergonomic design plays an

important role. Specialists are optimizing working processes and workflow layout to allow industries to be more productive. The layout of Intensive Care Units does not always seem to be designed with the same care for ergonomic and workflow conditions.

## EVITA REMOTE HELPS TO IMPROVE THIS SITUATION IN ONE IMPORTANT FIELD

Sometimes the layout of an ICU unit requires the preparation of a procedure on one side of the bed, then walking to the ventilator on the other side to hit the "alarm silence" or "suctioning" key, walk back to the other side to continue the procedure.

This can become a thing of the past. Evita Remote offers a convenient activation of routine functions within a radius of 3 m from the ventilator.



D-411-2010



D-412-2010



D-413-2010

## FUNCTIONS THAT CAN BE ACTIVATED THROUGH THE REMOTE PAD:

### Alarm silence

To silence an alarm while you are not close to the ventilator. For example, if you need to disconnect your patient for a short time to rearrange the breathing circuit.

### O<sub>2</sub> Suction

Preparing the catheter and activating the suctioning function of the ventilator from the same side.

### Manual / Insp. Hold

Allows a greater distance during x-ray use. Manual sigh breath e.g. after suctioning can be given without walking over to the ventilator.

### Exp. Hold

Stay in close contact with the patient to motivate him/her to cooperate with you while taking the measurement.

### Nebulizer

Prepare the nebulizer on the "non-ventilator" side of the bed and still be able to activate it without walking around the bed.

### Alarm Reset

Confirm a displayed alarm, or reset the original ventilation mode when Apnea ventilation was activated.

If you have a ceiling system, you can also have the remote pad put into the cable guide.

To connect a remote pad to an EvitaXL, Evita 4 or Evita 2 dura, a Multi-Function board (MF-board) is required. This board comes with a nurse call output as a standard.

## TECHNICAL DATA

Available keys	– Alarm silence – O <sub>2</sub> suction – Insp. Hold – Exp. Hold – Activate nebulizer – Alarm Reset
Cable length	3 m

## ORDER LIST

Option Evita Remote (factory installed)	84 14 477
Retrofit kit Evita remote	84 14 472
<b>Required hardware:</b>	
Option MF board / nurse call (factory installed)	84 14 475
Retrofit kit MF board / nurse call	84 14 476
<b>Required software:</b>	
EvitaXL SW 5.10 or higher	
Evita 4 / Evita 2 dura SW 4.0 or higher	

\*Order no. 84 14 240

### HEADQUARTERS

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany

www.draeger.com

### REGION EUROPE CENTRAL AND EUROPE NORTH

Dräger Medical GmbH  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
Tel +49 451 882 0  
Fax +49 451 882 2080  
info@draeger.com

### REGION EUROPE SOUTH

Dräger Médical S.A.S.  
Parc de Haute  
Technologie d'Antony 2  
25, rue Georges Besse  
92182 Antony Cedex, France  
Tel +33 1 46 11 56 00  
Fax +33 1 40 96 97 20  
dlmfr-contact@draeger.com

### REGION MIDDLE EAST, AFRICA, CENTRAL AND SOUTH AMERICA

Dräger Medical GmbH  
Branch Office Dubai  
Dubai Healthcare City, P.O. Box 505108  
Dubai, United Arab Emirates  
Tel + 971 436 24 762  
Fax + 971 436 24 761  
contactuae@draeger.com

### REGION ASIA / PACIFIC

Dräger Medical  
South East Asia Pte Ltd  
25 International Business Park  
#04-27/29 German Centre  
Singapore 609916, Singapore  
Tel +65 6572 4388  
Fax +65 6572 4399  
asia.pacific@draeger.com

### REGION NORTH AMERICA

Draeger Medical, Inc.  
3135 Quarry Road  
Telford, PA 18969-1042, USA  
Tel +1 215 721 5400  
Toll-free +1 800 437 2437  
Fax +1 215 723 5935  
info.usa@draeger.com

### Manufacturer:

Dräger Medical GmbH  
23542 Lübeck, Germany  
The quality management system at Dräger Medical GmbH is certified according to ISO 13485, ISO 9001 and Annex II.3 of Directive 93/42/EEC (Medical devices).